

FTS

Fentanyl Test Strip Instructions

1

FOR ALL TESTS



prep shot & put
needle aside



leave residue in
cooker or baggie

Meth & MDMA ONLY

2

1-2
oz.




drop cooker &
residue into
1 to 2 oz. of water

3



hold test strip
in cup for
15 seconds

4

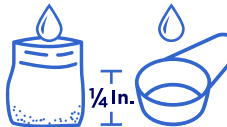

add **1/4 inch**
of water

5

do not
go past
**blue
line**




insert test strip
for **15 seconds**

INSTRUCTIONS CONTINUE ON THE OTHER SIDE ➡

FTS Fentanyl Test Strip Instructions (cont.)

FOR ALL TESTS

6

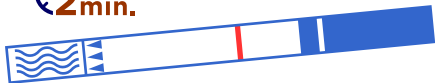

remove test and
lay flat for **2 minutes**



WARNING!

These test strips do not guarantee that your drugs are free of synthetic opioids - if you suspect contamination, **don't use them!**

**NEVER USE ALONE
ALWAYS CARRY NARCAN**

CHECK THE LINES

7

- **ONE UPPER** line on test strip - **POSITIVE** for fentanyl



- **TWO** lines on test strip - **NEGATIVE** for fentanyl

if lower line is very light, it is still a negative result



- **ONE LOWER** line or **NO** lines on test strip - test is **INVALID**



If your test strip is invalid, retest with a new strip